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Dear patient,

Please read these instructions for use carefully because they contain important information that you should pay attention to while using this medical device. If you have any questions, please consult your doctor or psychotherapist.

Instructions for use

Kalmeda®

Intended purpose:

Cognitive behavioural therapy for reducing the stress of chronic tinnitus.

How the app works:

The Kalmeda® Tinnitus App gives you fast, flexible access to therapy for chronic tinnitus, wherever you happen to be. This includes guideline-based Cognitive Behavioural Therapy (CBT) in digital form, along with relaxation exercises, as well as nature and other background sounds as audio files and knowledge about tinnitus.

The CBT is delivered as a structured exercise programme intended to help you recognise harmful attitudes towards your tinnitus and replace them with helpful ones. Alongside CBT based on the Zurich Resource Model, elements of Acceptance and Commitment Therapy (ACT) are also used. A central part of the therapy is clarifying values and life goals, from which specific commitments for action are then derived.

Providing knowledge on tinnitus and its treatment options is intended to promote a better understanding of the condition and help reduce the stress of tinnitus. In addition to CBT, you are provided with guided meditations as audio files as well as written instructions for two-minute relaxation exercises, intended to be integrated into your everyday life, with a reminder function available to prompt you.

Kalmeda® also includes nature and background sounds as MP3 files to provide acoustic distraction from chronic tinnitus (noiser principle). The files play automatically in continuous loops to provide a constant background sound.

The main goal of the therapy is to help you reduce dysfunctional control strategies by raising your willingness to cope with unpleasant sensations.

Minimum age:

The use of Kalmeda® is intended for patients aged 18 and over.

Contraindications and instructions:

- **Cognitive impairments:** If you have severe cognitive impairments or serious mental illnesses, such as dementia or acute psychosis, you should not use Kalmeda®, as it cannot be assumed that the therapy can be carried out continuously.
- **Visual impairment:** The use of Kalmeda® is not suitable for you if you suffer from a visual impairment that cannot be corrected with visual aids.
- **Jaw joint problems:** If you suffer from jaw joint problems, or if such problems occur during therapy – for example, craniomandibular dysfunction (CMD), teeth grinding (bruxism) or overstretching of the jaw – or if you have previously experienced dislocation (luxation) of the jaw, please consult your dentist or orthodontist before undertaking jaw joint exercises. Also pay attention to symptoms that may indicate a jaw joint disorder, such as restricted jaw opening, noises in the jaw joint like cracking, clicking or grinding, pain when opening the jaw or noticeable tension in the jaw joints. Improper use of the app may worsen existing complaints or lead to new ones.

Side effects:

There are no known side effects.

Note:


Please report any serious incidents to the manufacturer and to the Federal Institute for Drugs and Medical Devices, Medical Devices Division, via the website: www.bfarm.de

Directions for use:

Kalmeda® guides you through the programme with an easy-to-use menu and is structured in three successive stages:

- **Kalmeda Start:**

After downloading, you will find initial information about your condition, along with tips for treating the accompanying symptoms.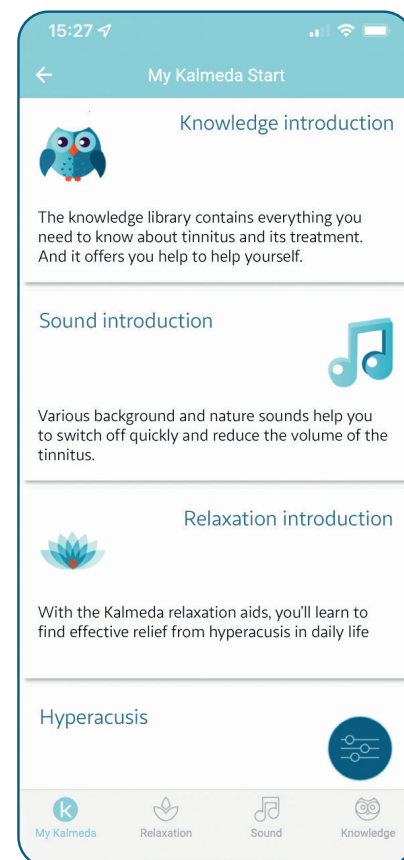


- **Kalmeda Go:**


This is the actual therapy section. The CBT section is divided into five levels, each with nine stages, and can be accessed under **MY TRAINING PROGRAMME**. Each level focuses on a specific theme:

1. Directing attention
2. Relaxation
3. Mindfulness
4. Acceptance
5. Self-efficacy




Each stage includes interactive self-reflection questionnaires, knowledge input, motivational aids, best-practice examples, and guidance for putting things into practice in your daily routines. As you progress through the training programme, you will get to know and reflect on your subconscious attitudes and behaviours. You keep emotional logs and set yourself individual, achievable goals in the areas of relaxation, mindfulness and acceptance, which are stored under **MY GOALS**, where they can be edited at any time.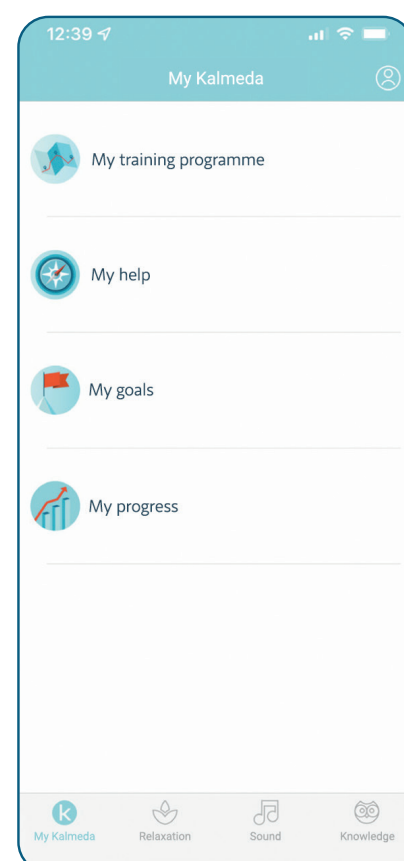

You are given various aids and will build up a support group for yourself. The self-defined aids and helpers are stored under **MY HELP** where they can be edited at any time. Aids and helpers form a kind of toolbox to help you achieve your self-set goals. At the beginning of each stage, you are asked about progress towards these goals, and further assistance is offered if needed.

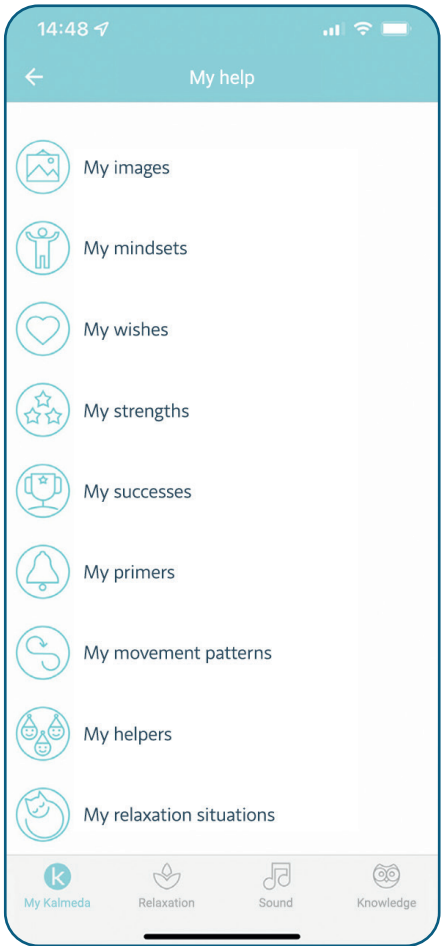
Alongside the exercise programme, calming nature and other background sounds can be accessed under **SOUND**. Under **RELAXATION**, you will find a wide range of relaxation exercises and guided meditations tailored to tinnitus.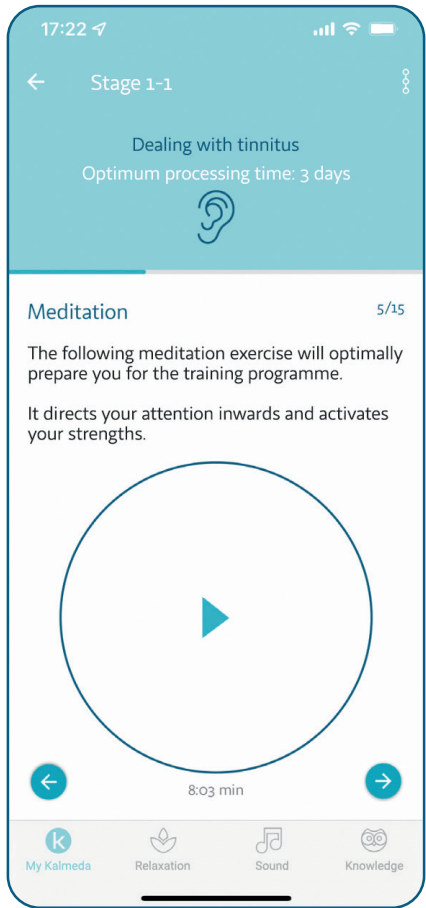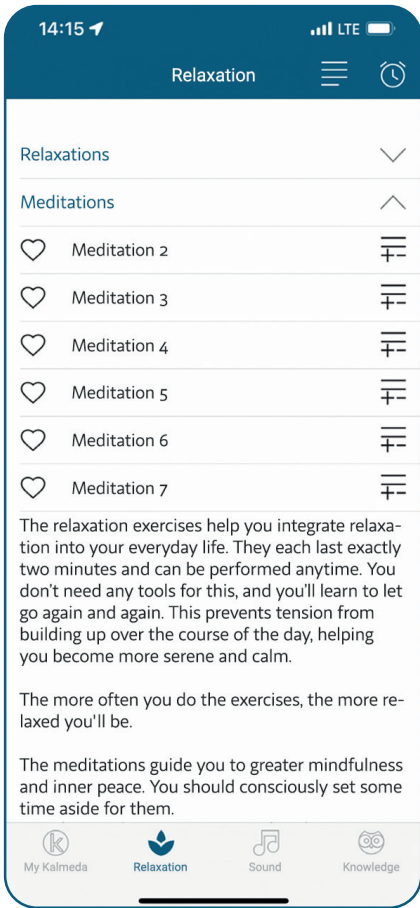
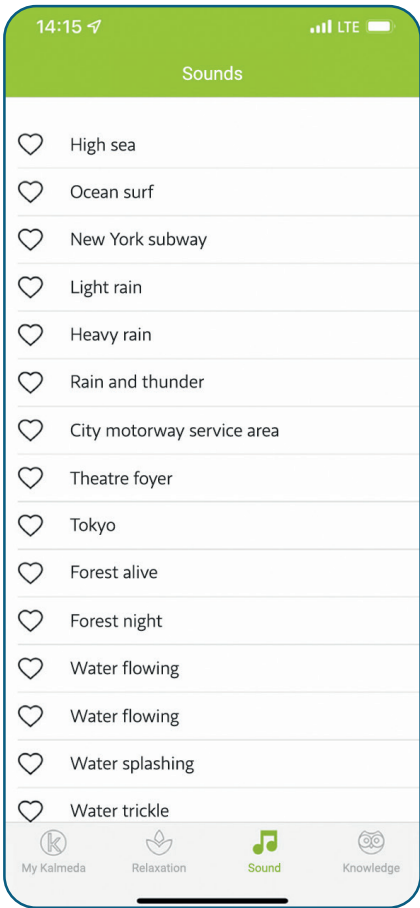


Under **KNOWLEDGE**, you can find information on various topics and terms related to tinnitus. You will get to know your body and condition better and will understand which measures can help improve your symptoms.

The support function also allows you to receive assistance with medical or technical problems or questions.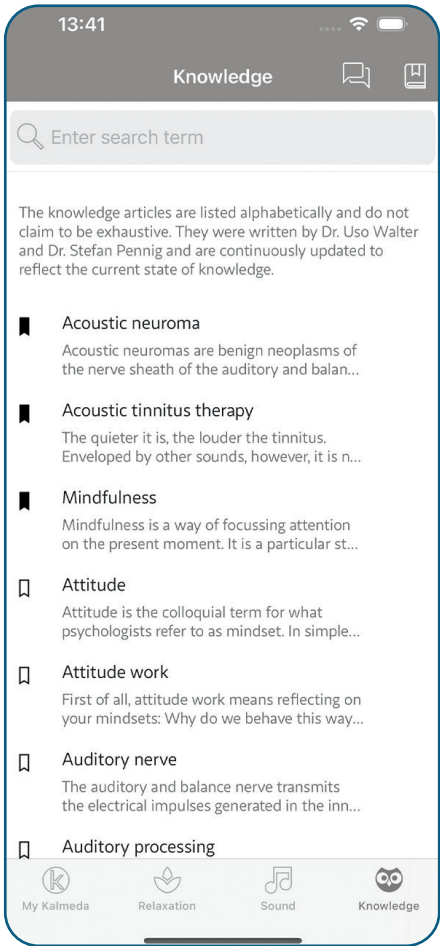

The progress of your therapy and usage intensity can be monitored independently under **MY PROGRESS**.

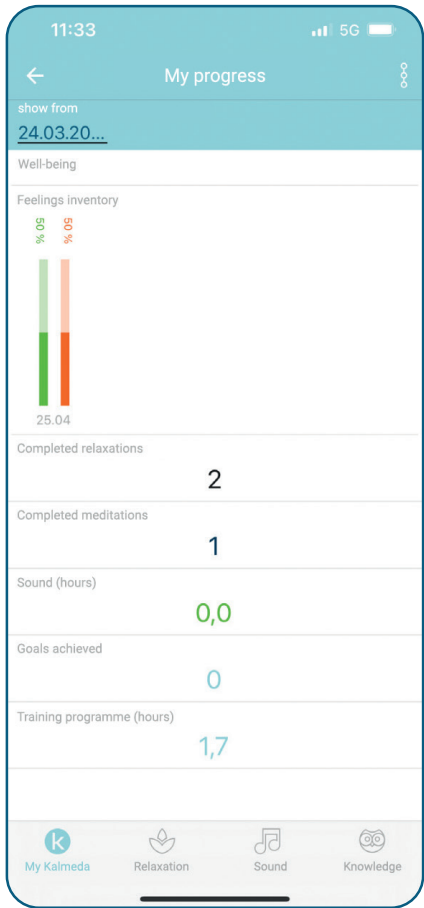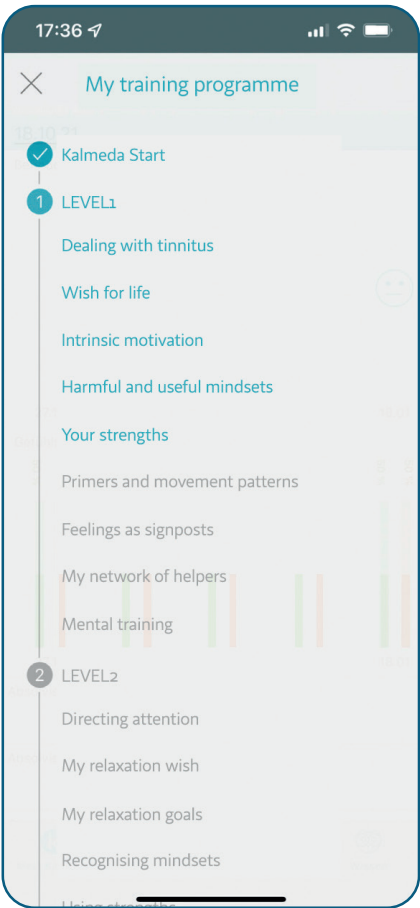- **Kalmeda Plus:**

After completing the 12-month training programme, you can continue to use the app to consolidate the success of the therapy.

Duration of use:

Kalmeda® should be used for 12 months to achieve long-term therapeutic success.

The individual stages of CBT each have minimum completion times, as progressing through them too quickly would prevent changes in attitudes and behaviours from being internalised and integrated into daily life. These time limits are an important part of the therapy, as they are intended to create lasting effects through repetition and consolidation, just as in face-to-face therapy.

Technical requirements:

Compatibility assurances

Minimum operating system versions:

Android from version 12

iOS from version 18

To use Kalmeda®, you need an internet-enabled device (smartphone or tablet) and a stable internet connection (mobile data or WiFi).

To use Kalmeda® to the best effect, the browser should be the latest version of Google Chrome (Android) or Apple Safari (iOS).

The current guidelines of the German Federal Office for Information Security (BSI) for password and communication encryption and authentication are observed in line with BSI TR-03161 (Requirements for Applications in Healthcare) and are monitored by information security management and updated as necessary. You are informed about the respective security setting options in the app.

Legal information:

You can find the General Terms and Conditions, the Legal Notice, the Privacy Policy and the consent forms under **MY ACCOUNT**.

If you need a printed version of our instructions for use, please contact our support team at info@pohl-boskamp.digital.

Pohl-Boskamp Digital Health GmbH
Kaddenbusch 4
25578 Dägeling, Germany
Phone: 04826 59 – 0
E-mail: info@pohl-boskamp.digital

Single Registration Number (SRN): DE-MF-000004969

Date of information: 2025-11

Manufacturer

Follow the electronic instructions for use

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